



## HOW TO CHANGE OUT 06500-LED OTOSCOPE BULB

**STEP ONE: Twist MacroView head clockwise to remove from otoscope handle**



**STEP TWO: Bulb located in the bottom portion of otoscope head (see arrow)**



**STEP THREE: Pull bulb out in direction of arrow below by pinching exposed portion of bulb with thumb and index finger (use or tweezers or similar tool)**



**STEP FOUR: Insert new bulb in chamber in direction of arrow as shown below**



[www.oaktreeproducts.com](http://www.oaktreeproducts.com)

800.347.1960

otp@oaktreeproducts.com