



Changing the Filter on the MedRx Ultra Vac

The particulate filter is located on the front panel for ease of service (see red arrow below). It should be replaced when significant amounts of debris are visible in it.



To replace the filter:

- disconnect filter from mount by pushing the clear cover up and rotating it to the left, then pull down
- change the filter element and rubber O-ring with a new one
- replace the cover by pushing the clear cover up and rotating it to the right

NOTE: The filter must be in place for the vacuum wand to operate properly. Should condensation develop in the particulate filter of the vacuum wand tube, run the system for 10 minutes to remove the moisture.