

This hearing screener allows for a basic assessment of an adult's hearing ability.

Persons having trouble hearing all four tones should be referred to a qualified hearing healthcare specialist for a reliable hearing assessment.

Operation:

- Use the screener in a quiet room.
- Turn on the power switch and wait 15 seconds until the power is activated.
- Disinfect the screener. Do not use alcohol.
- Hold the screener close to the person's ear and press the 500Hz button once.
- Let the tone play for a few seconds and register whether the person can hear it or not (e.g. tell the person to lift their hand if they can hear the tone). Repeat this process for each of the other three frequencies.
- Disinfect the screener and test the other ear.

Specification:

- Output: 40dB at 500 Hz, 1000 Hz, 2000 Hz and 4000 Hz.
- Power supply: 2 AAA size batteries.