

EZ-Adjuster Style Instructions

Materials used: foam, anti-fog plastic film, elastic strap, adjuster.

Step 1

Clean hands with soap and water or hand sanitizer.




Step 2

Remove a single ClearMask™ from packaging and peel off the protective layer on the outward-facing side of the mask. Inspect for any damages, rips, holes or contamination. Do not use product if any damages or contamination are found.



Step 3

Position the mask in your hand with the foam facing inward and the nose cutout on top, towards your fingertips. Bring the mask towards your face, such that your nose fits into the nose cutout and your chin rests on the bottom foam piece.

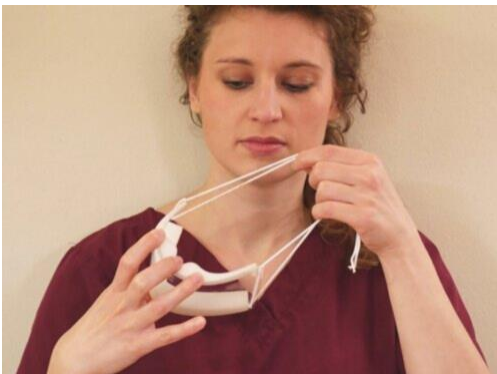

Step 4

With your other hand, take both the top and bottom straps and place them over your head. The bottom strap sits around the base of the head, below the ears. The top strap rests higher at the back of the head, above the ears. Do not crisscross straps.



Step 5

Use the EZ-Adjuster in the back to adjust the mask by pushing inward to tighten or pulling outward to loosen the straps.



Step 6

Ensure proper fit and direct contact between the face and the foam without gaps.

