




## teleloop amplifier

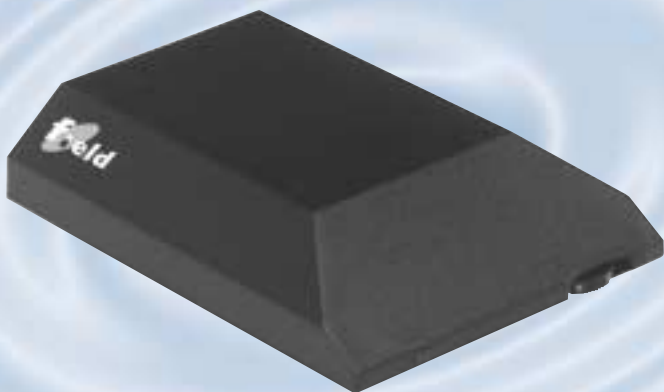

Ⓞ user guide

)))))) **Phonic Ear**<sup>®</sup>

**other resources online at [www.phonicear.dk](http://www.phonicear.dk)**

# **table of contents**

---

FIELD function . . . . .	1
FIELD features . . . . .	2
FIELD accessories. . . . .	3
FIELD instructions. . . . .	5
Troubleshooting . . . . .	10
Test and approval . . . . .	10
Warranty . . . . .	11

## **FIELD function**

---

FIELD is a teleloop amplifier. FIELD transfers the sound from TV and radio directly to your hearing aid (or telemagnetic receiver) and you receive the sound loud and clear.

# FIELD features

- ① loop connection  
To connect teleloop cable

- ② 12V AC connection  
To connect 12 volt power transformer

- ③ radio/TV connections  
To connect radio, television, etc.


- ④ microphone connection  
To connect external microphone

- ⑤ indicator light  
Shows that power is on


- ⑥ volume control  
To adjust volume level

- ⑦ bass control  
To adjust bass levels


- ⑧ treble control  
To adjust treble levels

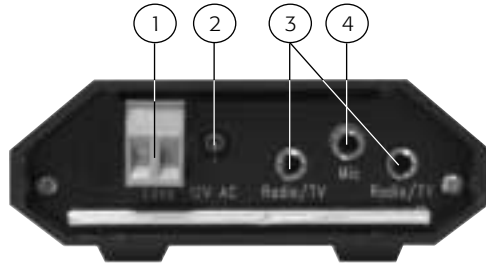

back

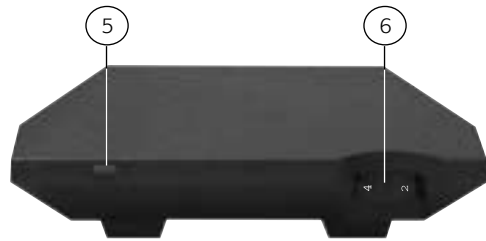

front

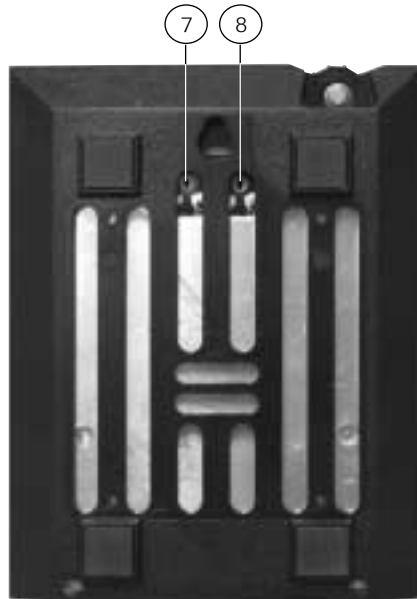


bottom

## FIELD accessories (included with FIELD system)

---


- 9 teleloop cable  
Installed in rooms where  
FIELD will be used

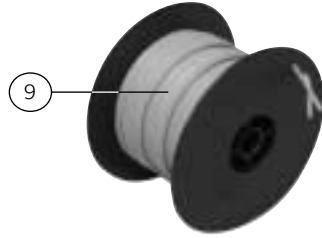

- 10 teleloop cable clips  
To secure teleloop cable




- 11 screwdriver  
To adjust bass and treble

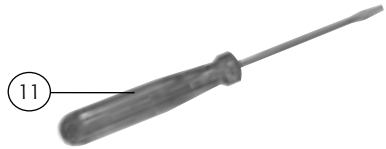


- 12 connector set  
(optional in some countries)  
Includes:

- Adaptor, SCART/phono
- Adaptor: 6,3mm (.25in) plug stereo to 3,5mm (.125in) stereo jack
- Connector cable: 2m (6.5ft), 3,5mm (.125in) plug stereo to 3,5mm (.125in) stereo plug
- Stereo cable: 2m (6.5ft), 3,5mm (.125in) stereo plug to 2 x phono plug

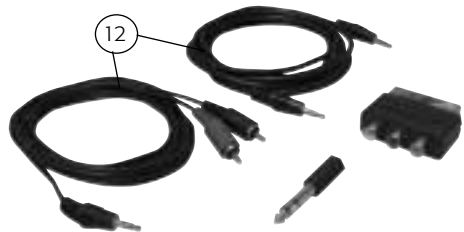

- 13 power transformer  
(optional in some countries)

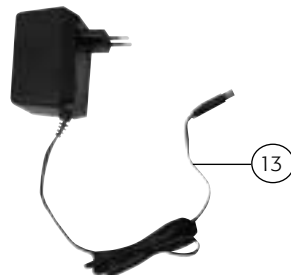

## FIELD accessories (not included, sold separately)

- ⑭ clip microphone  
Placed in front of source loud-speaker if sound source cannot be attached to teleloop amplifier by an output connection. Can also be used to amplify conversations.



- ⑮ boundary microphone




- ⑯ cushion loop



# FIELD instructions

## 1a teleloop installation

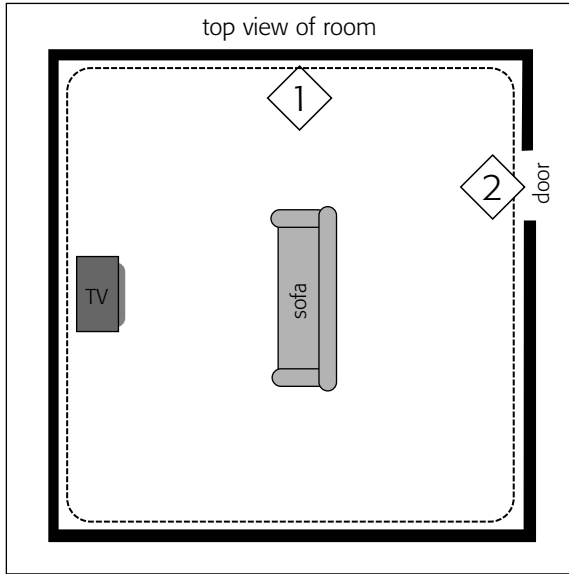

1 Place teleloop cable along wall, floor, or ceiling of room where FIELD will be used and secure with cable clips.

2 To avoid cable damage, run cable under carpet, behind baseboard or around doorframe.

**ATTENTION:** Make sure that teleloop cable is fully stretched out. If cable is too long, shorten it to fit room size.

## 1b cushion loop installation (optional)




Place cushion loop pad under a cushion in the seat where you are listening to the sound source.

The cushion loop limits the range of the teleloop and possible “interference ability”. It can also be easily moved around the room if necessary.

## FIELD instructions

### 2 connecting teleloop cables

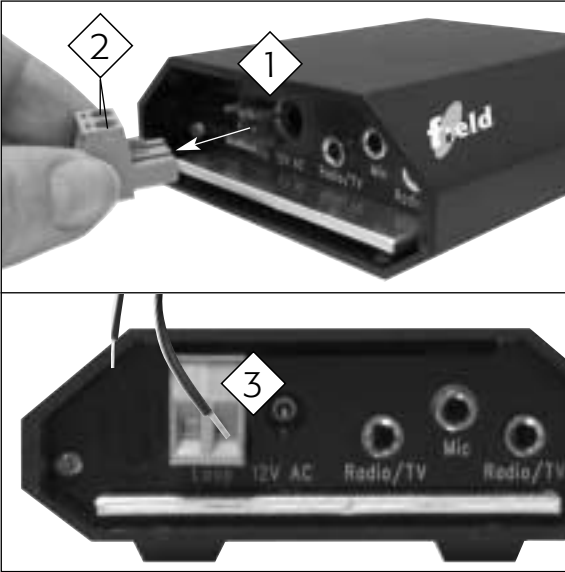

1 Remove **Loop** connector from back of unit.

2 Loosen cable connection screws.

3 Strip ends of cable to expose 1 cm of wire.

Insert ends of cable into connector holes and tighten screws using the screwdriver.

Replace connector into **Loop** hole on unit.

### 3 connect to TV



Plug one end of the cable (from connector set) into one of the **Radio/TV** connections on back of FIELD.

Then plug other end of the cable into **Line Out** or **SCART** connection on TV.

**NOTE:** Use only cables and jack plugs supplied by an authorized Phonic Ear representative/installer or FIELD may be damaged.



## FIELD instructions

---

### 4 connect to radio



Plug one end of the cable (from connector set) into one of the **Radio/TV** connections on back of FIELD.

Then plug other end of the cable into **Line Out** connection on radio.

**NOTE:** *Sound sources other than TV and radio can be connected to FIELD. Contact an authorized Phonic Ear representative/installer for more information.*

*Use only cables and jack plugs supplied by an authorized Phonic Ear representative/installer or FIELD may be damaged.*

### 5 connect to microphone (optional)



If sound source cannot be attached to teleloop amplifier by an output connection, then microphone can be placed in front of the source loudspeaker.


Plug microphone cable into **Mic** connection on back of FIELD.

**NOTE:** *When clip microphone or boundary microphone are used for conversations, the TV and radio should be switched off.*

*Use only cables and jack plugs supplied by an authorized Phonic Ear representative/installer or FIELD may be damaged.*


## FIELD instructions

### 6 connecting transformer

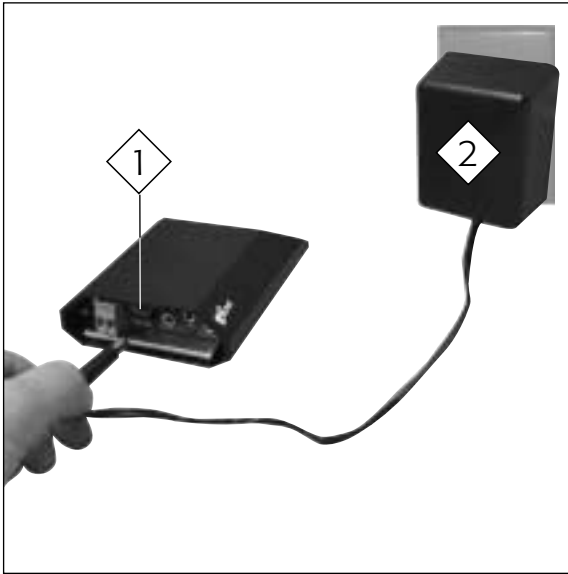

- 1 Plug transformer cord into **12V AC** connection on back of FIELD.
- 2 Plug transformer into wall socket. Red light on front of FIELD should go on.

### 7 adjusting volume

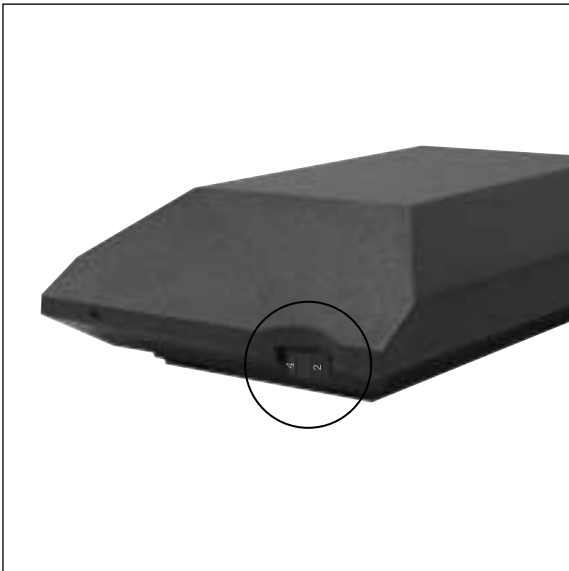

Use volume control on front of FIELD to adjust volume level:

Min = Lowest volume


Max = Highest volume

**NOTE:** *As a general rule, the volume should be adjusted so that you don't experience irritating changes in volume when you change between "M" and "T" positions on your hearing aid.*

# FIELD instructions

---

## 8 adjusting bass and treble

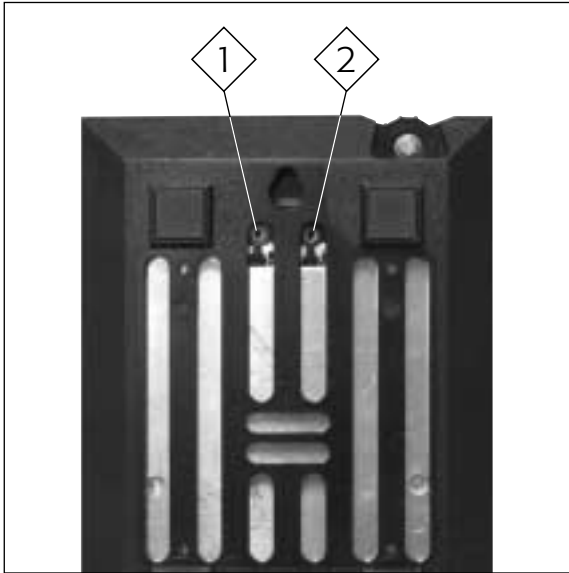

1 Turn left control on bottom of FIELD clockwise to increase bass level and counterclockwise to reduce bass level.

2 Turn right control on bottom of FIELD clockwise to increase treble level and counterclockwise to reduce treble level.

**NOTE:** Use the screwdriver included with FIELD to adjust bass and treble controls.

## 9 hearing aid setting



Set hearing aid to “T” position when using teleloop.

## troubleshooting

---

Should your FIELD teleloop amplifier not function satisfactorily, please check the list below:

### **The red light on the front of FIELD unit does not come on**

- Check to see that the wall contact is switched on.
- Switch off the wall contact for 10 seconds and then switch it on again.

### **The red light on the front of FIELD comes on, but there is no signal when the TV or radio is turned on**

- Check that the volume is correctly adjusted. See [7].
- Check that all the connections are connected properly.
- Check that your hearing aid is set to the T-position (telecoil). See [9].

### **The signal from FIELD sounds distorted**

- Check that the volume is not too high. See [7].

If you still have trouble after trying the ideas above please contact your local authorized Phonic Ear representative/installer (see information on back cover).

## test and approval

---

FIELD is approved and CE marked in accordance with the relevant directives.



*Waste from electronic  
equipment must be  
handled according to  
local regulations.*

# warranty

---

A one-year limited warranty is issued to the original consumer of this product(s) by Phonic Ear (two years inside the EU).

## **time period of warranty**

This warranty will go into effect upon the date of original purchase of this equipment, and will remain in effect for one calendar year (two years inside the EU) from that date as long as the instrument remains the property of the original consumer.

## **what is covered by this warranty**

Any electronic component which, because of workmanship, manufacturing, or design defects, fails to function properly under normal use during the life of this warranty will be replaced or repaired at no charge for parts or labor when returned to the factory service center. Transportation is paid by the customer. If it is determined that repair is not feasible, the entire unit may be replaced with an equivalent unit upon mutual agreement of the manufacturer and customer.

## **what is not covered by this warranty**

- 1** Malfunctions resulting from abuse, neglect, or accident.
- 2** Peripheral accessories as itemized within the product brochure, when such items are returned after 90 days from the original purchase.
- 3** Alkaline batteries (if applicable).
- 4** Instruments connected, installed, used, or adjusted in a manner contrary to instructions provided by the manufacturer.
- 5** Consequential damages and damages resulting from delay or loss of this instrument. The exclusive remedy under this warranty is strictly limited to repair or replacement as herein provided.
- 6** Products damaged in transit unless investigated by the shipper and returned to the warrantor with the investigation report.

Phonic Ear reserves the right to make changes in the design or construction of any of its instruments at any time without incurring any obligation to make any changes whatsoever on units previously purchased. This warranty is in lieu of all other expressed warranties. All expressed and implied warranties will terminate upon the expiration of this written warranty. No representative or person is authorized to represent or assume for us any liability in connection with the sale or use of our products other than as set forth above.

The above warranty does not affect any legal rights that you might have under applicable national legislation governing sale of consumer goods. Your distributor may have issued a warranty that goes beyond the clauses of this limited warranty. Please consult your distributor for further information.

## **what to do if you need service**

If you require service under the terms of this warranty, carefully package the instrument to prevent damage and return it to your distributor (see information on back cover). Please provide your receipt, a detailed description of the problem, your full name, billing address, and telephone number.

Your local representative:



---

**international:** + 45 3917 7101

**u.s.:** 800.227.0735, then press 7 • **canada:** 800.263.8700

**website:** [www.phonicear.dk](http://www.phonicear.dk)